Regolamento (UE) 2017/745
EN 14683:2019+AC2019
Tipo IIR, filtrazione batterica >98%

CARA., CARA.KIDS,
CARA.COMFORT, CARA.COMFORT
KIDS

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DESCRIPTION: Non-sterile three-layer medical mask: breathable outer layer, high-filtering middle layer, durable and hypoallergenic inner layer.

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WARNINGS AND LIMITATIONS OF USE: Medical masks protect the patient from infectious agents and the wearer from splashing potentially contaminated liquids. They may also be intended for use by patients and others to reduce the risk of spreading infections, particularly in epidemic or pandemic situations.

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The masks used are considered potentially contaminated; it is therefore essential that:

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• The body of the mask is not touched by the fingers/hands of the wearer.

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• Hands are disinfected after removal of the mask

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• The mask worn covers the wearer's nose and mouth.

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• A used face mask should be disposed of when no longer needed or between procedures.

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It is important to wear and dispose of the mask correctly, otherwise it can be a source of infection itself. Surgical masks are not personal protective equipment, so they are not intended to protect the wearer's respiratory tract from environmental contaminants.

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STORAGE AND TRANSPORT: Check that the product is still within its shelf life before use. Store and transport the unused device in its packaging in a dry (<80% humidity), clean place and at a temperature between -20°C and +25°C. Keep out of direct sunlight.

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DISPOSAL: Disposable masks used in a professional environment (hospitals, clinics, etc.) must be disposed of as hazardous waste in accordance with national laws. Products used outside the professional environment can be disposed of as unsorted waste.

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HOW TO WEAR THE MASK:

Before putting on the device, check that the packaging is intact and the product has no defects, tears or damage, and make sure your hands are clean.

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• Make sure that both flaps are completely unfolded.

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• Put the mask on by putting the elastic bands behind your ears.

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• Cover your mouth and nose with the mask, making sure it fits tightly around your face.

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